

# Hana<sup>®</sup> Distraction Pad

Post-Free Patient Positioning




# Post-Free with the Hana Distraction Pad

Increase utilization of your Hana® Surgical Table with the latest post-free techniques. Introducing the **Hana® Distraction Pad**, the only surgical table manufacturer validated solution for Post-Free hip distraction that couples optimum patient safety and stability with the freedom to create a pressure free environment at the perineum.

## Improving hip preservation surgery by promoting advanced post-free techniques

- Patient transfer - no draw sheet required for patients weighing up to 450 lbs (207 kg)
- Ergonomic patient set up and positioning
- Allows full functionality of the Hana in the supine position
- Optional Perineal Post usage for additional stability

Designed to simplify patient transfer from a transport gurney to the surgical table

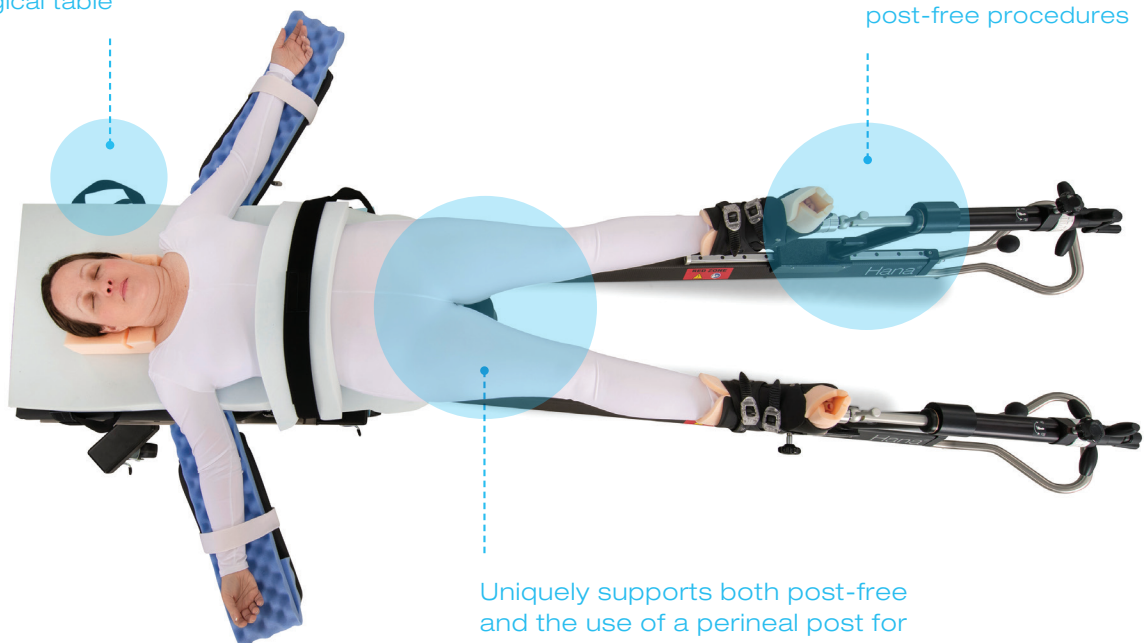


Uniquely supports both post-free and the use of a perineal post for post-free hip distraction friction



## Safe Transfers. Secure Positioning. Post-Free Surgery.

When handles are lifted, the smooth center sled remains in contact with the surface for easy patient set up and repositioning. The reinforced backing allows for patient weight of 450 lbs (207 kg) to be transferred without tearing the soft foam. When the patient is placed on the Hana Distraction Pad, the high friction surface conforms to the body and minimizes patient movement under traction.

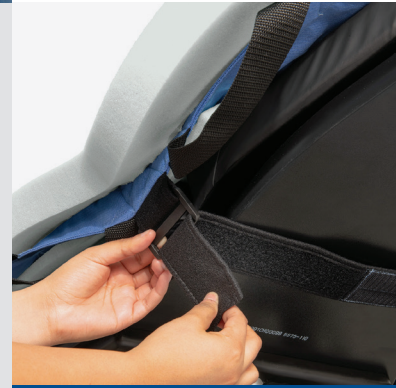


450 lbs

weight capacity (207 kg)

## Optimized Safety

A perineal post may be inserted after patient transfer to safely stabilize the patient. Use the tearaway feature to expose the perineal post receptacle.

Based on patient's anatomy, weight and traction resistance, a perineal post may be used to prevent unintended movement.

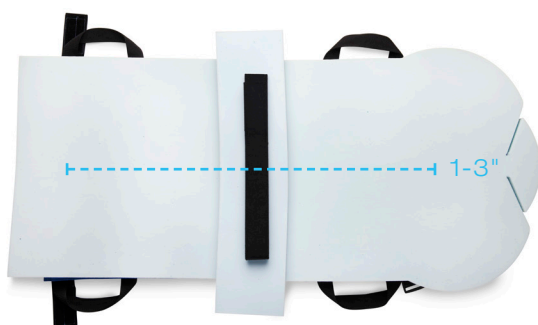


## Patient Positioning

Center the patient medially on the pad and 1-3" superior to the perineal post cutout to enhance pelvic stability.

# Components, Procedures & Ordering Information

## Pad Components

1. Integrated transport/positioning handles
2. Center sled
3. Grip strips
4. Tear-away for perineal post exposure
5. Table securing straps

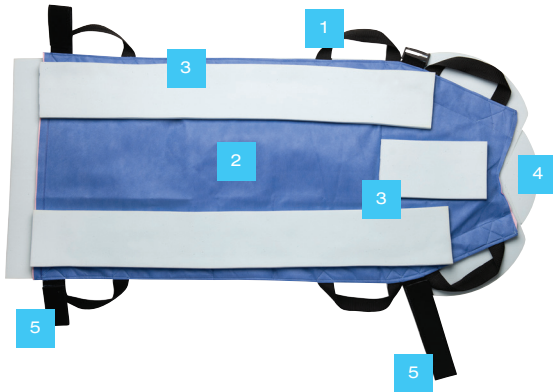

## Supported Procedures

- Hip Arthroscopy
- Anterior Hip Replacement
- Hip Fracture Fixation
- Femur and Tibia Fracture Repair

## Compatible Tables/Devices

- Hana® Orthopedic Surgery Table
- Hana® SC Surgery Center Table
- ProFx® Pelvic Reconstruction Orthopedic Fracture Table
- Hana SSXT® Specialty Surgery Table Extension

## Ordering Information

Case Number	Kit Description	Case Qty.	Distraction Pad w/Safety Strap	Leg Spar Ball Joint Protectors	Adjustable Arm Board Pads	27" Soft Strap, Armboard	Universal Disposable Boot Liners	Supine/Lateral Headrest	Foot Pedal Cover
6855DPAA-5	Anterior Approach to Total Hip Arthroplasty	5/cs	•	•	•	•	•	•	•
6855DPHA-5	Hip Arthroscopy	5/cs	•	•	•	•	•	•	
6855DPB-5	Basic Traction	5/cs	•	•	•	•			
6855DP-5	Hana Distraction Pad with Safety Strap	5/cs	•						
6855DPS-10	Hana Distraction Pad Safety Straps	10/cs							



30031 Ahern Avenue  
 Union City, CA 94587-1234 USA  
 Telephone: 510-429-1500  
 Toll Free: 800-777-4674  
 Fax: 510-429-8500  
 Outside USA: +1-510-429-1500  
[mizuho.com](http://mizuho.com)  
[newhipnews.com](http://newhipnews.com)

Distributed Exclusively by Mizuho OSI

Patents Pending. Note: Mizuho OSI is constantly improving its products. All specifications are subject to change without notice. Mizuho OSI is a Delaware Corporation. Manufactured in the USA and Mexico.