

Specifications & Components

Specifications

Frame Weight

Model 5321G	40 lbs. (18 kg)
Model 5319G	36 lbs. (16 kg)

Lateral Adjustment of Pads	0"–6" (0–15 cm)
Intraoperative Flex of Pads for Lordosis Control	3.5" (8.8 cm)

Patient Weight Capacity	300 lbs. (136 kg)
-------------------------	-------------------

Individual Patient Care Kits

The patient care kits help protect the frame while promoting proper positioning and patient hygiene.

Kit Includes:

2 pad sleeves, 1 transverse bar pad, 1 base cover drape, 2 TLC™ Arm Cradles and either a ProneView® Cushion Insert Headrest Pillow or GentleTouch® Pillow.

REF 5322 PV Wilson Plus™
Patient Care Kit with ProneView (6/cs)

REF 5322 Wilson Plus
Patient Care Kit with GentleTouch (6/cs)

Ordering Information

REF 5321G Wilson Plus
Radiolucent Frame with Tempur-Pedic Medical Pads,
Spinal Surgery Top

REF 5319G Wilson Plus
Radiolucent Frame with Tempur-Pedic Medical Pads,
Universal Design



30031 Ahern Avenue
Union City, CA 94587-1234 USA
Telephone: 510-429-1500
Toll Free: 800-777-4674
Fax: 510-429-8500
Outside USA: +1-510-429-1500
mizuho.com
newhipnews.com



Emergo Europe
Prinsessegracht 20
The Netherlands



Tempur-Pedic Medical Pads

The Tempur-Pedic medical replacement pad set makes it easy to insure high quality surfaces at all times.

REF 5319-37 Set of 2

Optional Accessories



Spinal Surgery Top

10604	Patient Safety Strap with Orange Aid® Gel Pad, secures patient to Wilson Plus
5322 PV	Wilson Plus Patient Care Kit with ProneView (6/cs)
5322	Wilson Plus Patient Care Kit with GentleTouch (6/cs)
6946	Tempur-Pedic® Medical Positioning Pillow (Standard)
6947	Tempur-Pedic Medical Positioning Pillow (Queen)


Imaging Top

5855-550	Patient Safety Strap
10604	Patient Safety Strap with Orange Aid Gel Pad, secures patient to Wilson Plus
5322 PV	Wilson Plus Patient Care Kit with ProneView (6/cs)
5322	Wilson Plus Patient Care Kit with GentleTouch (6/cs)
6946	Tempur-Pedic Medical Positioning Pillow (Standard)
6947	Tempur-Pedic Medical Positioning Pillow (Queen)



Wilson Plus™

Radiolucent frame for posterior surgeries

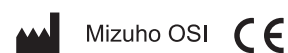

Note: Mizuho OSI is constantly improving its products. All specifications are subject to change without notice. • Tempur-Pedic® is a registered trademark of Tempur-Pedic North America, LLC • Mizuho OSI is a Delaware Corporation. Manufactured in the USA.

2017 ©Mizuho OSI **REF** NW0420 REV H

Convenient. Versatile.

The **Wilson Plus™** Radiolucent Frame provides a convenient and stable method of maintaining patients in a flexed position for Laminectomy, Decompression, Disc Surgery and Microdiscectomy procedures. Made with carbon composites and high impact plastics, the Wilson Frame delivers unrestricted radiolucency at the operating site.

The Wilson Plus Radiolucent Frame comes in two convenient models. One model is designed to fit on the Mizuho OSI® modular spine frames, allowing for maximum hip flexion and minimum lordosis when used with the leg sling. This model may also be used in conjunction with the 180° degree rotation capabilities of the Trios® and the Modular Table System (MTS) to safely position a patient without lifting.

The Wilson Plus universal design can be easily mounted on any general surgery table but is ideally suited for use on the Mizuho OSI Trios and MTS Imaging Tops.



Wilson Plus™ Features


300 lbs.
weight capacity (136 kg)

A. Easy To Use Crank System

The removable crank allows adjustment while the built-in torque limiter protects the unit from over-cranking.

B. Lordosis Control

Pads flex to adjust spinal lordosis and open the disc spaces for improved access at the surgical site.

C. Tempur-Pedic® Medical Pads

Tapered for optimal comfort, two full length Tempur-Pedic Medical Pads provide continuous support while adjusting laterally up to 10 inches (25.4 cm). This design aids in improved ventilation and reduced pressure on the abdomen.

The pads' Tempur construction evenly distributes the pressure load and contour to the patient's anatomy, providing enhanced support.

D. Convenient Transfer and Storage

Wilson Plus frames roll from storage to OR with minimal effort.

- Model 5321G includes a storage cart (shown)
- Model 5319G includes a roller case

E. 360° Degree Radiolucency

Wilson Plus frames offer 360° degrees of unobstructed radiolucency for easily obtainable images with either C-arm or X-ray.

